


Checking the IBP Pressure Sensor used in Tafonius, VitalStore and Sentinel monitors

The DPT-200 (DPT-400 in North America) pressure sensor looks like this:

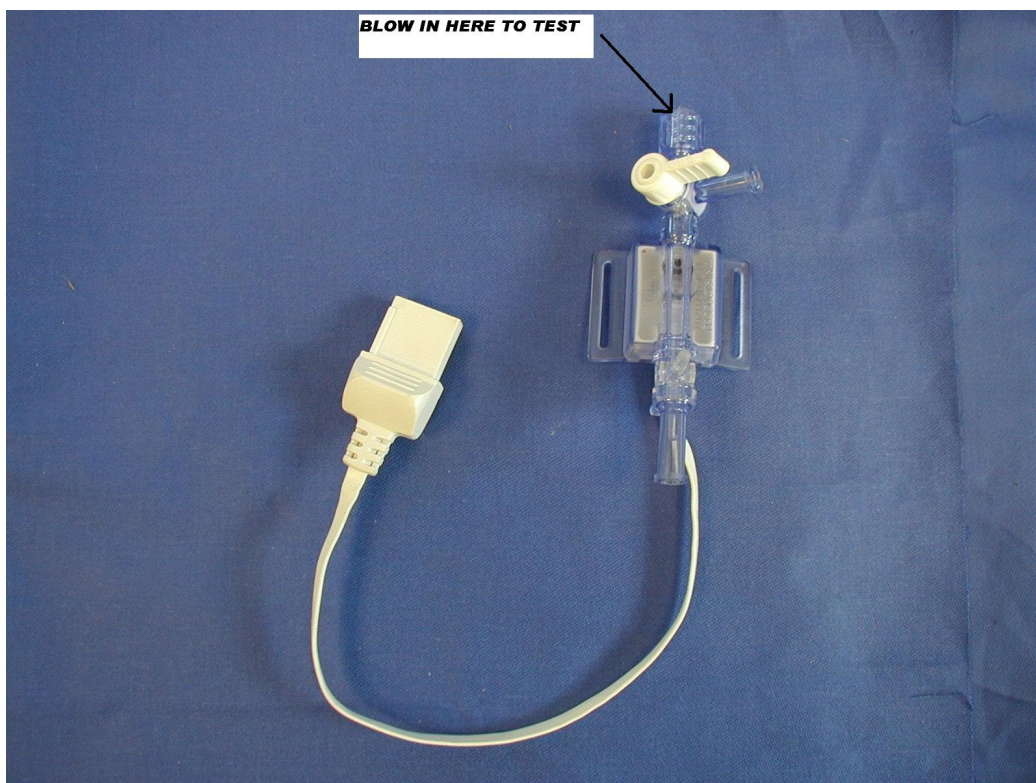

- 1) Connect the sensor to the IBP cable
- 2) Do not connect any fluid lines to the sensor
- 3) With nothing connected to the pressure ports, do a reset in the unit that you are using to monitor the trace
- 4) Blow into one of the open ports on the sensor, or use a small 2ml syringe to create some pressure. With the sensor tap in the position shown above, blow or use a syringe to create pressure in the end connection.

You should see a pressure trace on the screen of your monitor.

If you don't see a trace, do another reset and try again or try another sensor.