

# Retrieving Tafonius Log Data

## ***Event Log***


- Exit Tafonius
- From the Start menu click Control Panel->Administrative Tools->Event Viewer. (You can access the start menu by clicking on the windows key on the keyboard. )
- In the left-hand panel under “Event Viewer (Local)” Click on “Application” so that it becomes highlighted
- Click on the “Action” menu option and “Save Log File As”.
- If you've got a memory stick plug this in to one of the free USB ports on the ELO, give the file a name and then save the file to a location on the stick.

Please send this file to Vetronic Services Limited via an email.

## ***Slow Data Log***

### ***Export Data***

From the shell click on the Export Data button which opens a browser to allow you to choose the data to export. All data in the browser is listed in date order with the most recent files at the top of the browser.

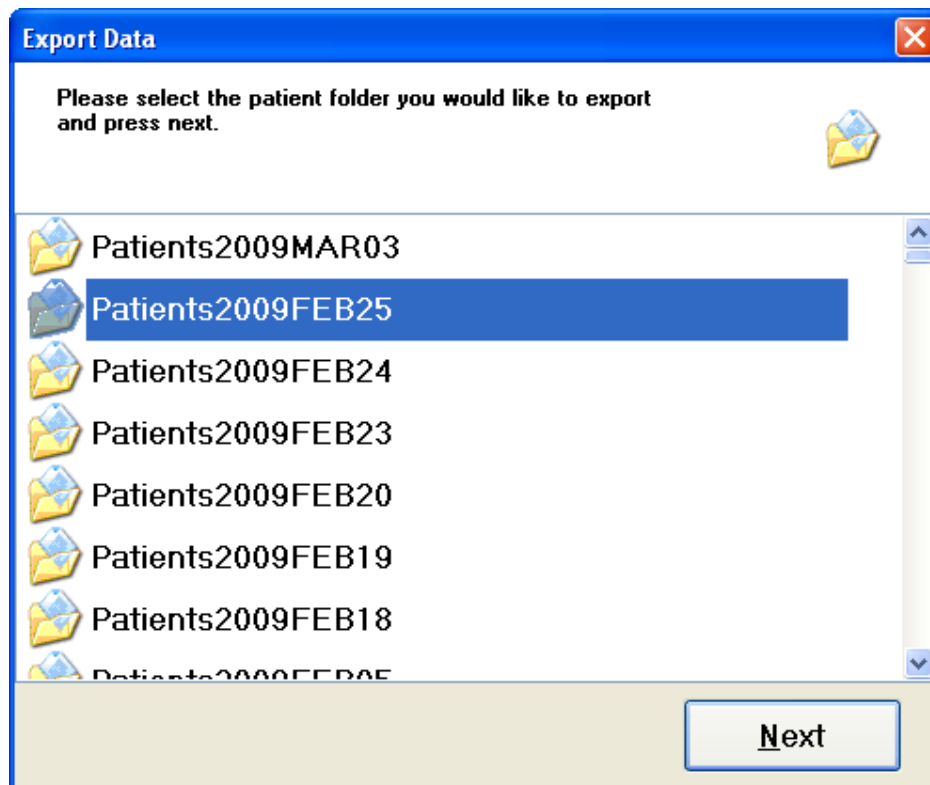

First select the patient that you wish to export data for and then click or touch the Next button. Then choose your destination drive (usually a USB flash drive named D:\) and touch or click the Export button. The Slow Data and the Anaesthetic Record data will be copied to your chosen destination.